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INSTRUCTIONS FOR USE IOVIR® Nasal spray with Carragelose®

What is IOVIR® Nasal spray with Carragelose®?

IOVIR® Nasal spray can be used supportively for the prophylaxis and treatment of flu-like illness caused by viruses. IOVIR® Nasal spray can be used additionally for the preventive and sustainable moistening of the nasal mucus membranes. IOVIR® Nasal spray can be used in adults and children from 1 year. IOVIR® contains **no preservatives**. A sterile filter guarantees the sterility of the solution even after repeated use. IOVIR® has no decongestant effect.

What does IOVIR® Nasal spray with Carragelose® contain?

IOVIR® Nasal spray is a clear, colorless, viscous solution without odour. 1 ml of the solution contains 1.2 mg iota-carrageenan (Carragelose®), 0.4 mg kappa-carrageenan, sodium chloride and purified water. Carragelose® is a natural active ingredient derived from marine red algae.

What are the properties of IOVIR® Nasal spray with Carragelose®?

Low humidity and air pollution can lead to a dry nasal mucous membrane. However, the nose is only able to fulfil its filtering function if the nasal mucous membrane is sufficiently moistened. IOVIR® Nasal spray forms a moisturising protective film on the nasal mucous membrane. This liquid layer serves as a physical barrier against external influences. The sustained moistening can guard against common cold and flu-like illness caused by common cold viruses.

The nose is the most common route of infection for viruses that trigger common cold and flu-like illness. Flu-like illnesses are caused by viruses such as rhinoviruses and coronaviruses for example. If viruses have penetrated into the nose, they can attach themselves to the mucous membrane, where they multiply and spread. This results in the typical symptoms of a cold, like a blocked or runny nose, sore throat and/or headache, cough, etc. In the early stages of a disease you often cannot distinguish between a common cold and influenza. Influenza is a potentially serious disease. The definitive diagnosis requires a doctor.

IOVIR® Nasal spray contains Carragelose®, a marine substance derived from red algae. Carragelose® has a dual positive effect.

- 1) IOVIR® Nasal spray forms a protective barrier against cold viruses. Common cold and flu-like illnesses are caused by a large variety of viruses, examples are rhinoviruses and coronaviruses. Carragelose® non-specifically envelopes cold viruses and prevents their adhesion to mucous membranes. Thereby, the reproduction and spreading of viruses responsible for infecting the airways is reduced.
- 2) IOVIR® Nasal spray forms a moisturising protective film. Due to the action of Carragelose®, this film remains longer on the nasal mucosa and serves as a physical barrier against external influences. This physiological moisture film supports the body's natural defences against cold viruses.

Therapeutic indications. What is IOVIR® Nasal spray with Carragelose® used for?

- 1) Can shorten the duration of flu-like illnesses and alleviate the severity of cold symptoms.
- 2) Can reduce new infections and resurgence of virus-related cold symptoms.
- 3) Is indicated for the prophylaxis and supportive treatment of viral infections of the respiratory tract.
- 4) Can be used additionally for the preventive and sustainable moistening of the nasal mucosa and thus for the prophylaxis and supportive treatment of all complaints resulting from dry and irritated nasal mucosa.

Dosage and method of administration

In what way and how frequently is IOVIR® Nasal spray with Carragelose® used?

At first signs of a flu-like infection (e.g., throat pain, tingling or runny nose, etc.), 1 puff should be sprayed at least 3 times daily into each nostril. Treatment should begin as early as possible after the first symptoms of a cold occur and needs to be continued until the symptoms have subsided. If fever develops or there is no improvement after 1 week of treatment, a physician should be consulted. For moisturising dry nasal mucosa, spray few puffs into the nose several times a day depending on individual requirements.

Remove the protection cap and pump a few times if necessary until a uniform atomised spray is discharged. IOVIR® Nasal spray is now ready for use. To prevent injury, insert the spray head carefully and not too deep into the nose! Hold the bottle upright when spraying. Replace the protection cap after use.

When should IOVIR® Nasal spray with Carragelose® not be used?

In case of known hypersensitivity to any of the ingredients.

Possible side effects of IOVIR® Nasal spray with Carragelose® . Interactions with other products

At present, no other side effects besides very rare cases of hypersensitivity are known. There are no known interactions. Studies on interactions with other products were not performed.

Special precautions for safe use

IOVIR® Nasal spray has a local action. Carragelose® is not absorbed by the cells of the nasal mucosa. IOVIR® Nasal spray can be used during pregnancy and breast-feeding. If the symptoms persist, you must consult a doctor. **For hygienic reasons and to avoid the transmission of pathogens, one spray bottle should always be used by one and the same person.** Keep out of the reach of children!

Further information

IOVIR® Nasal spray may be used for a longer period of time. There is no potential for addiction or risk of overdose. IOVIR® Nasal spray contains the natural substance Carragelose, a marine substance derived from red algae. IOVIR® Nasal spray is a medical device and clinically tested. IOVIR® Nasal spray contains no analgesic, no preservative, no constituent of animal origin or greenhouse gas. IOVIR® Nasal spray is lactose-free and **gluten-free and can be used by diabetics and vegans.**

Read instructions for use

Package size

20 ml bottle with spray pump; 1 puff = 0.14 ml of solution (142 puffs in total).

Shelf life

36 months. Due to the special filter system, the solution remains sterile even after repeated use.

IOVIR® PLUS Nasal spray can be used for 12 months after first use but not beyond the expiry date.

Storage instructions: Store between 5 °C and 30 °C  Protect from light 

Manufacturer: Hälsa Pharma GmbH, Maria-Goeppert-Straße 5, D-23562 Lübeck, Germany

Distributor:

Cube Pharma & Nutrition, 54 Menandrou str., Athens 10431, Greece, Tel. +30 210 5240963,

Fax. +30 210 5240754, www.cube-pharmaceuticals.gr, www.facebook.com/Cube.Pharma

Emergency Poison Center: +30 210 77 93 777

Date of revision

August 2017

0482

STERILE A

XXXX-YYYY-BP-XY